





WHAT is it?

The Bug is a low-profile shoe insert designed to control mild to moderate low tone pronation with an optimal amount of support and comfort, yet minimal volume in the shoe.

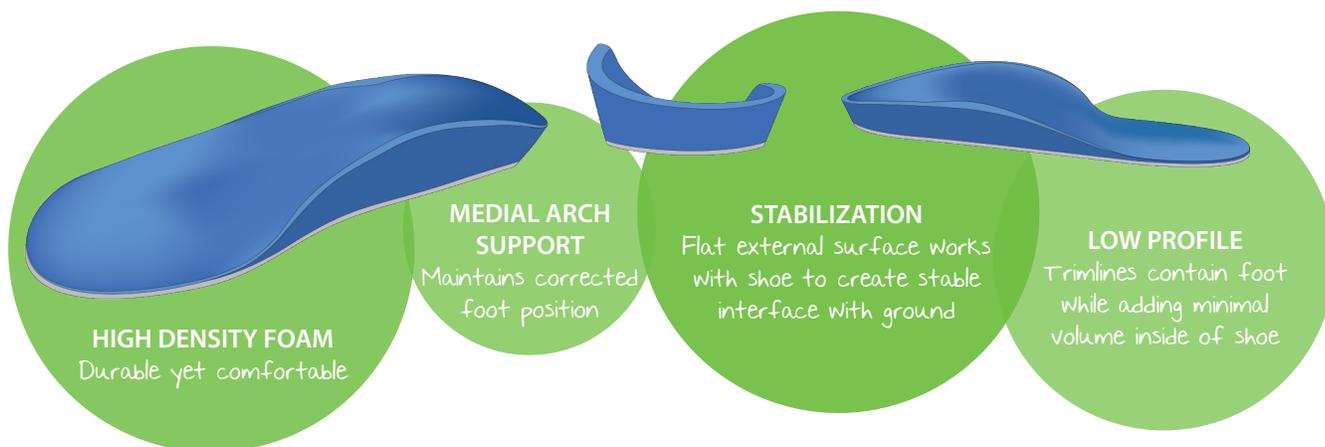

WHO is it for?

The Bug is designed to correct mild to moderate pronation.

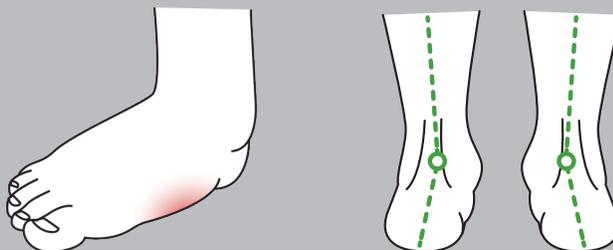


SIGNS OF PRONATION:

- Reduced medial arch
- Hindfoot eversion
- Forefoot abduction

HOW does it help?

The Bug, partnered with a supportive shoe, promotes a more comfortable and neutral gait by raising the medial arch and bringing the heel and forefoot to neutral. The internal plantar surface contours support the foot in a good position of function, and the flat external surface works with the shoe to provide a stable base of support.

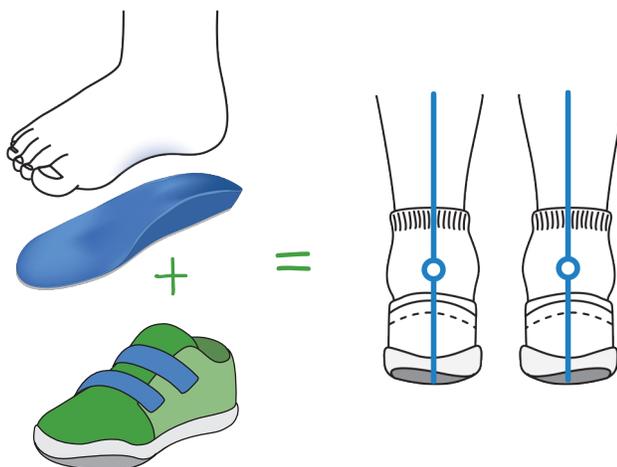

SHOE CHECKLIST:

- Straps or laces
- Removable insole
- Flexible sole
- Supportive



No
Added
Cost!


NEW SIZING PROCESS!

Thanks to the new optimized design, it's easy to choose the right size Bug insert! Just measure the insole of your current well-fitting shoe.


1 Remove the shoe's insole



2 Measure the insole's length in inches.

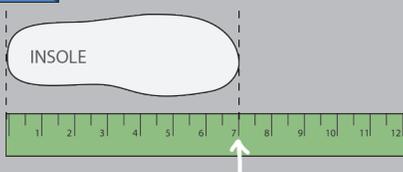

3 Order the Bug in the same size as the shoe's insole. Available in sizes 4.00-13.00. If in-between sizes, go up .25" in size.

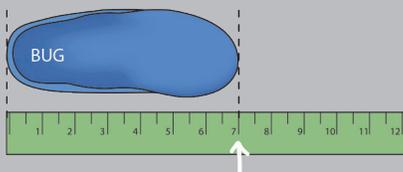


OPTIMIZATIONS

Modifications can be made to the Bug in order to help the following patient presentations:


Excess Pronation

A medial forefoot posting or full medial wedge can help counteract excess pronation by stabilizing the foot in a varus position.

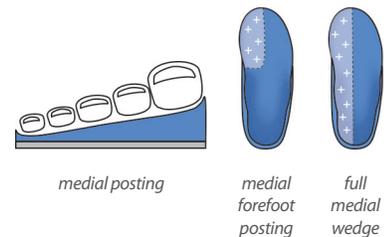

Leg-Length Discrepancy

A full-length lift, a full-length wedge, or a wedge tapered to the forefoot can help raise the height of one foot to discreetly accommodate the discrepancy.

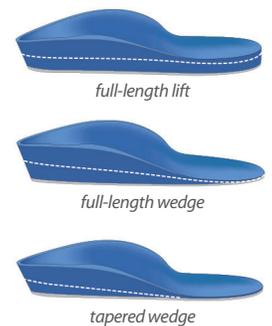

TO ORDER

Contact Customer Service at 800.848.7332 or customerservice@dafo.com

We want to hear from you! Send in a comment card or call us - we're looking forward to your feedback.