
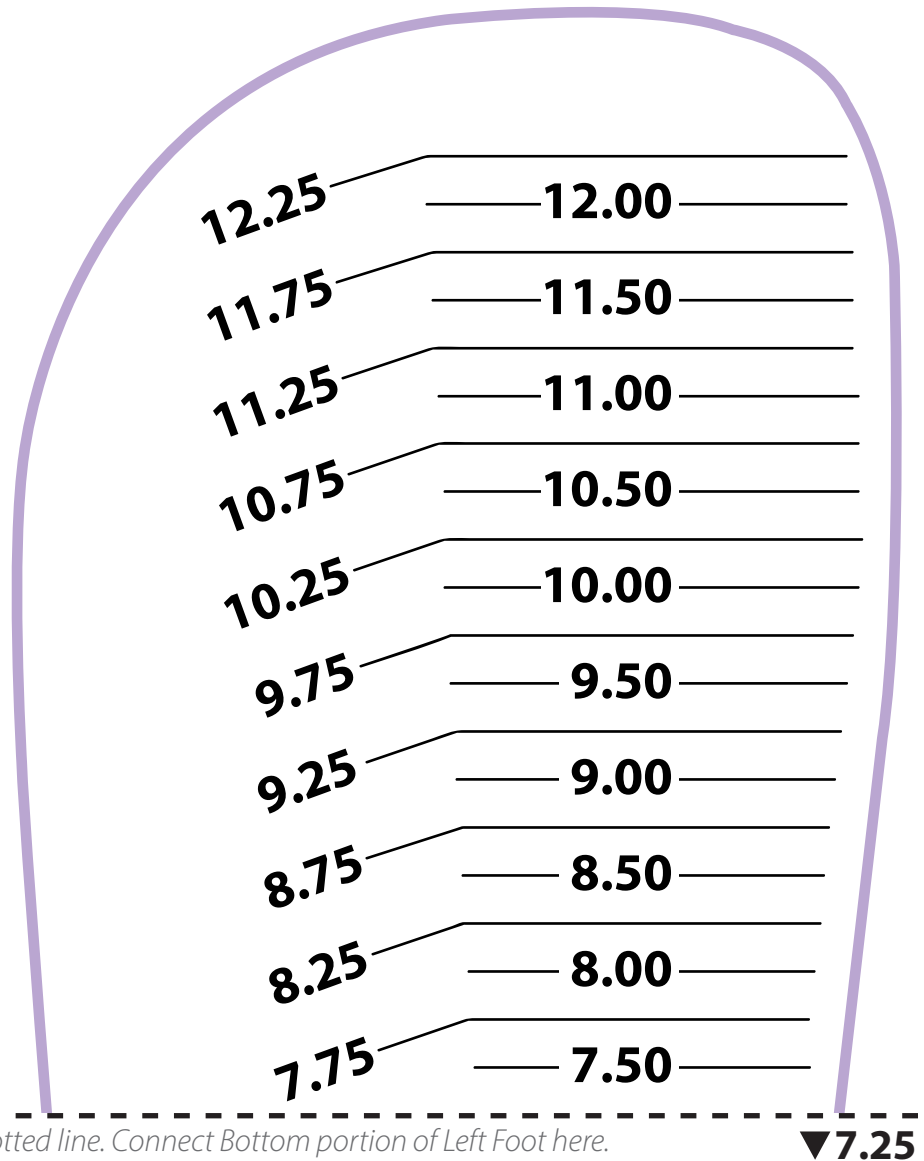


Fast Fit[®] Sizing Jig | Printable Version

Left Foot – Top

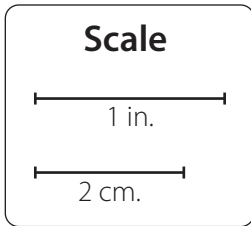

Questions?

Contact our **Customer Support** team:
800.848.7332 | customersupport@dafo.com
Monday–Friday, 6:00am–5:00pm (PST)

1 Print

To ensure accurate scaling and proper sizing, use the following printer settings:

1. With the Printable Sizing Jig pdf open in Adobe Reader, go to the File menu.
2. Click the Print sub-menu item.
3. Make sure scale is set at 100%.
4. Print.
5. After printing, verify that the scale on this sheet is accurate with a standard ruler.

2 Assemble

1. Follow the printing instructions (to the left). If the scaling is accurate, continue on.
2. Cut/fold along the dotted lines on the jigs' Top and Bottom portions.
3. Tape the Top and Bottom portions of the Left and Right sizing jigs at their corresponding dotted lines.

3 Size

1. Make sure the patient is wearing socks and have him/her stand on the jig. Align the outermost part of the heel and the inside of the foot with the designated lines on the jig.

**See Figures 1 and 2 above.*

2. Choose the length that will allow ¼ to ½ inch (.64 to 1.3 cm.) for growth.
3. Unless the patient's foot is significantly narrow, we recommend ordering a Wide size.

Fast Fit® Sizing Jig | Printable Version

Left Foot – Bottom

Cut/fold along dotted line. Connect Top portion of Left Foot here.

▲7.50



Figure 1: Outermost part of heel

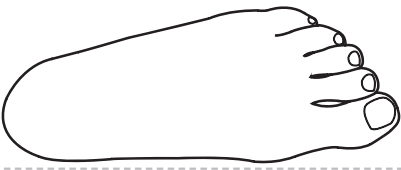


Figure 2: Inside of foot

7.25 — 7.00 —
 6.75 — 6.50 —
 6.25 — 6.00 —
 5.75 — 5.50 —
 5.25 — 5.00 —
 4.75 — 4.50 —
 4.25 — 4.00 —

Line inside of foot along this side of jig. (See figure 2.)


Sizes currently available
 (0.25 in. increments—Wide and Narrow)

PattiBob	4.00–12.25 in.
Chipmunk	
Cricket	
JumpStart Leap Frog	
JumpStart Bunny	4.00–9.00 in.
JumpStart Softback	
JumpStart Kangaroo	

**FAST
FIT**

LEFT

Scale


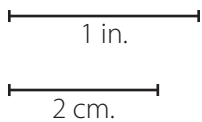


Line outermost part of heel along this line. (See figure 1.)

Printable Sizing Jig-outline — Left Bottom rev.04 (March 2024)

Fast Fit[®] Sizing Jig | Printable Version

Right Foot – Top

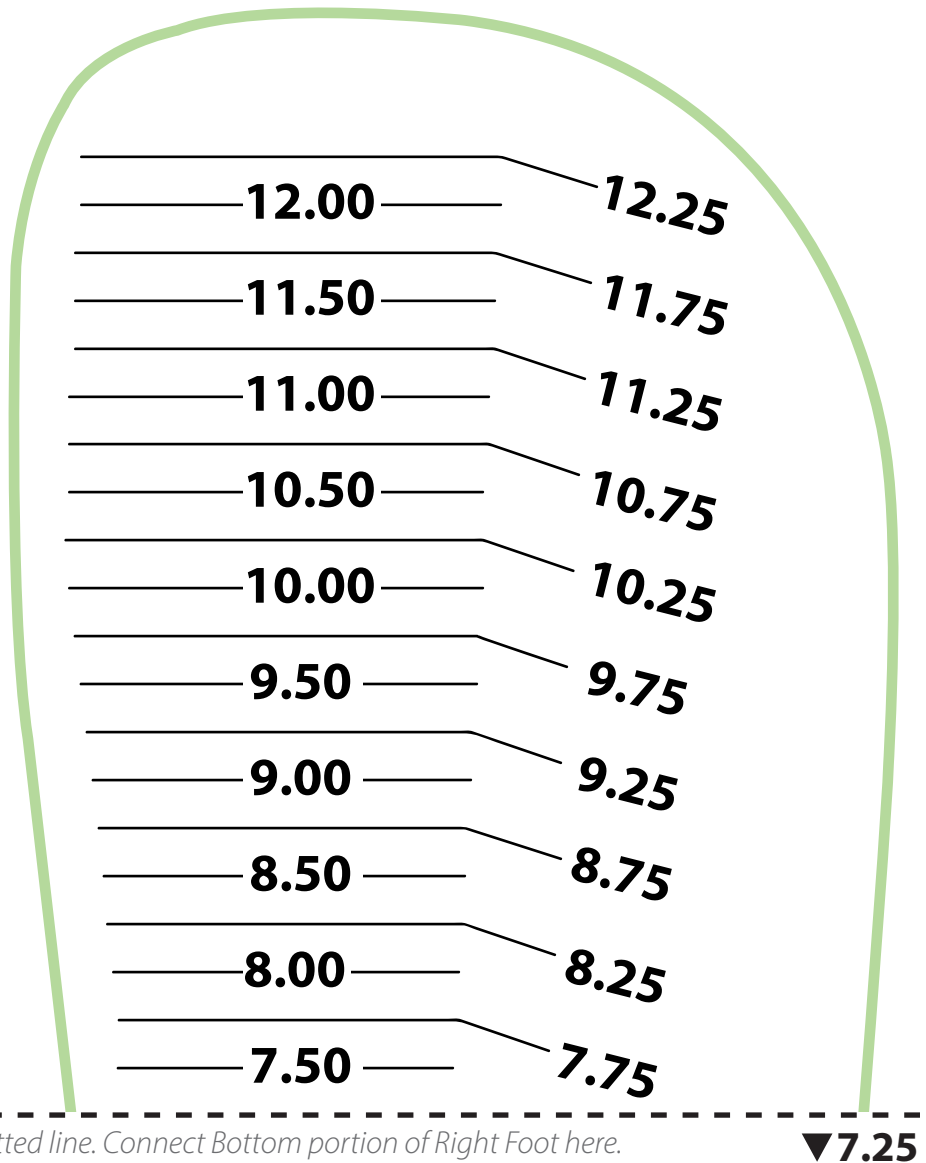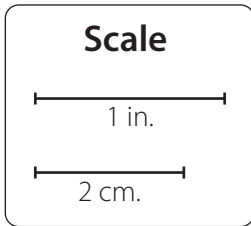

Questions?

Contact our **Customer Support** team:
800.848.7332 | customersupport@dafo.com
Monday–Friday, 6:00am–5:00pm (PST)

1 Print

To ensure accurate scaling and proper sizing, use the following printer settings:

1. With the Printable Sizing Jig pdf open in Adobe Reader, go to the File menu.
2. Click the Print sub-menu item.
3. Make sure scale is set at 100%.
4. Print.
5. After printing, verify that the scale on this sheet is accurate with a standard ruler.

2 Assemble

1. Follow the printing instructions (to the left). If the scaling is accurate, continue on.
2. Cut/fold along the dotted lines on the jigs' Top and Bottom portions.
3. Tape the Top and Bottom portions of the Left and Right sizing jigs at their corresponding dotted lines.

3 Size

1. Make sure the patient is wearing socks and have him/her stand on the jig. Align the outermost part of the heel and the inside of the foot with the designated lines on the jig.

**See Figures 1 and 2 above.*

2. Choose the length that will allow ¼ to ½ inch (.64 to 1.3 cm.) for growth.
3. Unless the patient's foot is significantly narrow, we recommend ordering a Wide size.

Fast Fit® Sizing Jig | Printable Version

Right Foot – Bottom

Cut/fold along dotted line. Connect Top portion of Right Foot here.

▲7.50

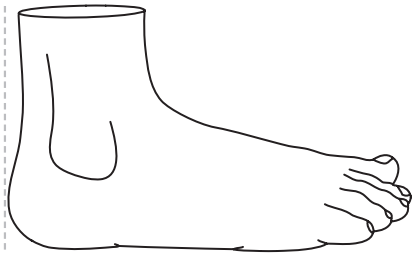


Figure 1: Outermost part of heel

Figure 2: Inside of foot




Line inside of foot along this side of jig. (See figure 2.)

- 7.00 — 7.25
- 6.50 — 6.75
- 6.00 — 6.25
- 5.50 — 5.75
- 5.00 — 5.25
- 4.50 — 4.75
- 4.00 — 4.25

Sizes currently available

(0.25 in. increments—Wide and Narrow)

PattiBob	4.00–12.25 in.
Chipmunk	
Cricket	
JumpStart Leap Frog	
JumpStart Bunny	4.00–9.00 in.
JumpStart Softback	
JumpStart Kangaroo	

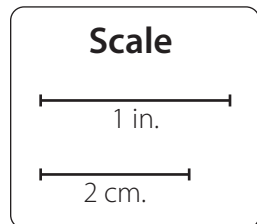

RIGHT

Line outermost part of heel along this line. (See figure 1.)

Printable Sizing Jig-outline — Right Bottom rev.04 (March 2024)